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EN USAGE

The **ARNOMED DISPENSER ALU S-500** unit is used for hand disinfection and sanitising with disinfectants. The dispenser is suitable for metering disinfectants, soaps, and liquid lotions. The dosage can be adjusted manually per pump shot.

To do this, the pump regulator must be set to the left for 1.4 ml, in the middle for 2 ml and to the right for 1 ml for each pump burst.

Included in delivery

- Disinfectant dispenser (a)
- Dosing pump (b)
- 3x screws (c)
- 3x dowels (d)
- Back panel for mounting (e)
- 500 ml Euro bottle (f)
- Operating instructions

- ❶ You will need a Phillips screwdriver and a drill for mounting.

Safety instructions

Recommended height of the dispenser: 120 cm.

Before mounting, check the wall condition and ensure the wall is suitable for plastic plugs. Consider that the dispenser must be easily accessible for use and refilling. There should be at least 15 cm of space above the dispenser to slide the dispenser onto the wall bracket. Leave at least 20 cm of space below the dispenser.

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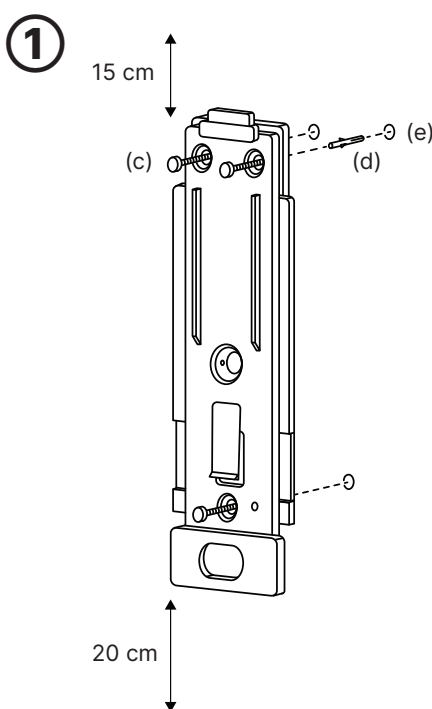
EN ASSEMBLY INSTRUCTIONS

- 1 Mark fixing holes through the back panel (e) with a pencil, drill holes, insert dowels (d) and tighten screws (c) with a Phillips screwdriver.

- 2 Hold the dispenser (a) above the back wall (e) against the wall and slide it down until it locks into place.

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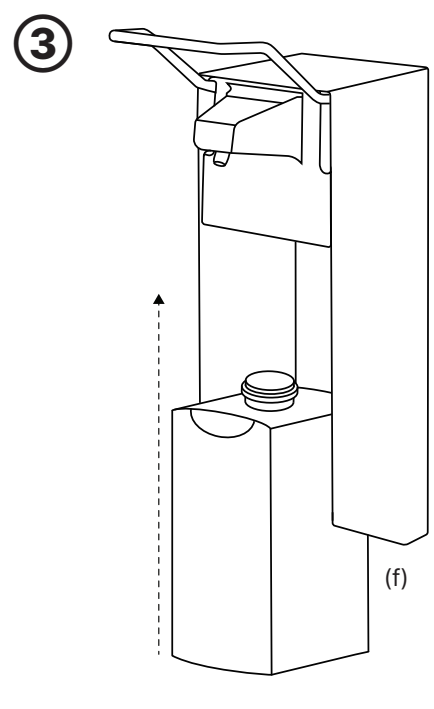
EN ASSEMBLY INSTRUCTION

- 3 Insert the bottle (f) into the dispenser (a) from below.

- 4 To use the dispenser (a), hold your hand under the dispenser and operate the arm lever.

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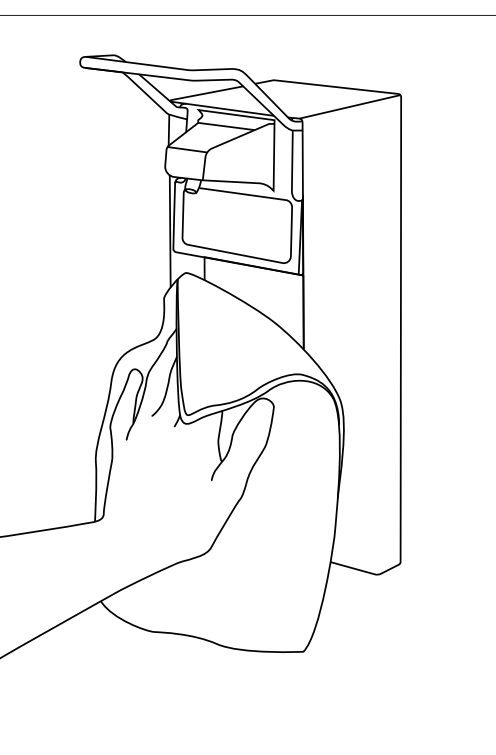
EN MAINTENANCE

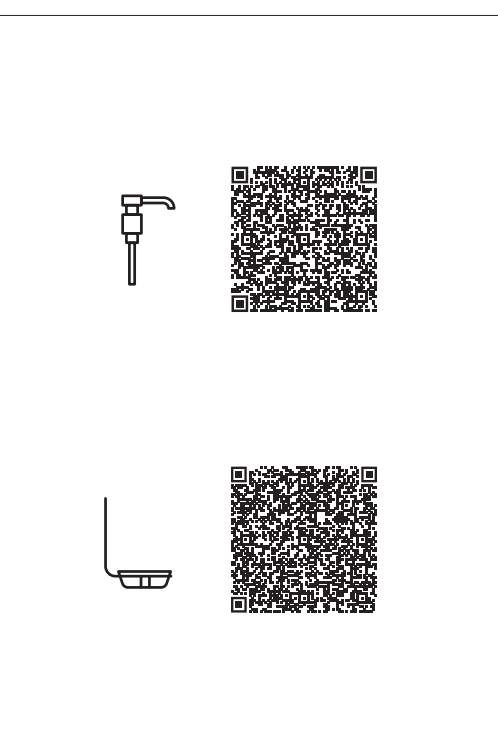
For cleaning, wipe the dispenser with a cloth and water and disinfect the casing with a suitable surface disinfectant. When cleaning, follow the recommendations of the Robert Koch Institute (www.rki.de/EN). The **ARNOMED DISPENSER ALU S-500** is not dishwasher safe.

EN GUARANTEE

The statutory guarantee rights apply. Wear and tear parts of the disinfection dispenser, such as the dosing pump and the Euro bottle, are excluded from the manufacturer's warranty.

EN DISPOSAL

When disposing of the disinfection dispenser, ensure it complies with regulations and that the components are recycled.

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EN REPLACEMENT PARTS

As the pump dispenser may wear over time, please use the QR code to directly find a suitable and inexpensive spare part from your manufacturer.

❶ Optional: Drip tray

You can attach a drip tray to avoid stains on floors or work surfaces when using the disinfectant dispenser. Find suitable drip trays directly from your manufacturer via the QR code.

Please don't hesitate to contact us for any questions or concerns: ARNOWA GmbH, Kugelbreite 30, 33154 Salzkotten, Germany | www.arnowa.de | info@arnowa.de

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