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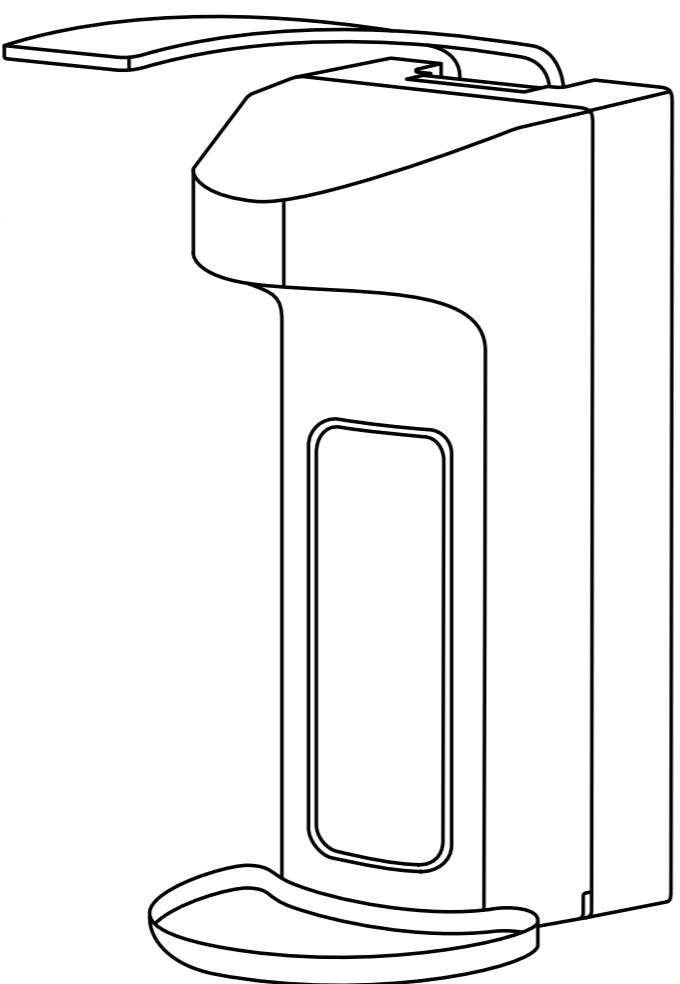
EN USAGE

The ARNOMED DISPENSER ABS 500 unit is used for hand disinfection and sanitising with disinfectants. The dispenser is suitable for dispensing disinfectants, soaps, and liquid lotions. The ARNOMED DISPENSER ABS 500 can be attached to the wall or set up freely. The dosage can be adjusted manually per pump shot.

To do this, the pump regulator must be set to the left for 1,4 ml, in the middle for 2 ml and to the right for 1 ml for each pump shot.

Included in delivery

- Disinfectant dispenser (a)
- Dosing pump (b)
- 3x screws (c)
- 3x screws (d)
- Back panel for mounting (e)
- 500 ml Euro bottle (f)
- Drip tray (g)
- Operating instructions (h)
- Key (i)

● You will need a Phillips screwdriver and a drill for mounting.

Safety instructions

Recommended height of the dispenser: 120 cm.

Before mounting, check the wall condition and ensure the wall is suitable for plastic plugs. Consider that the dispenser must be easily accessible for use and refilling. There should be at least 15 cm of space above the dispenser to slide the dispenser onto the wall bracket. Leave at least 20 cm of space below the dispenser.

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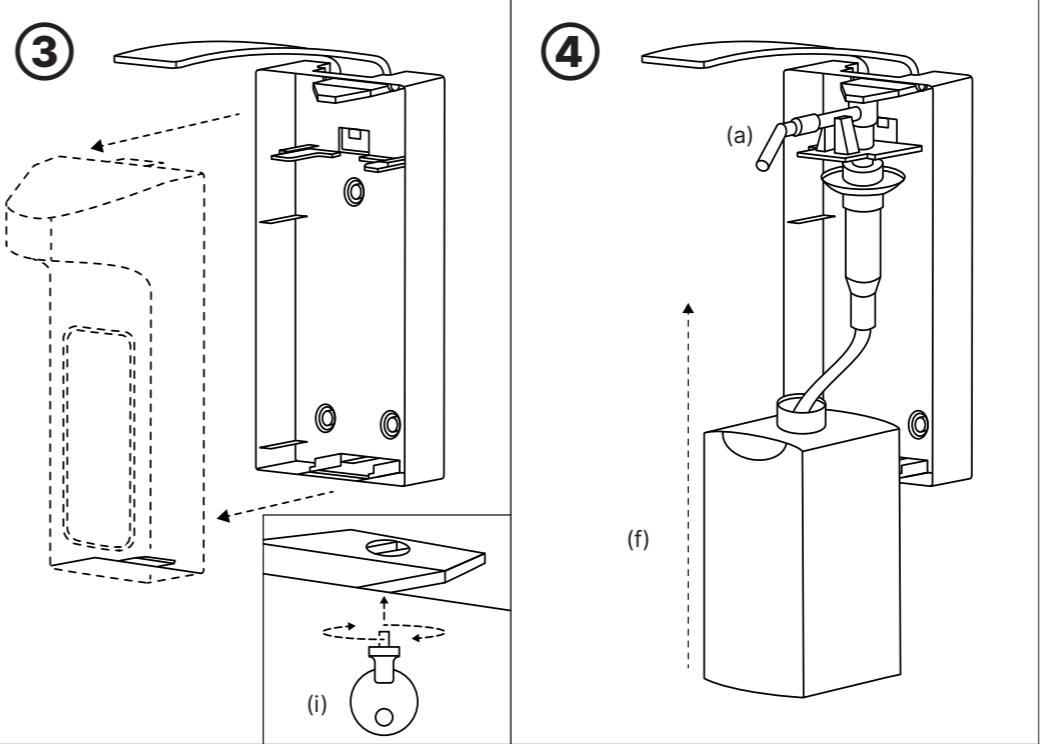
FR INSTRUCTIONS DE MONTAGE

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①

Mark fixing holes through the back panel (e) with a pencil, drill holes, insert dowels (d) and tighten screws (c) with a Phillips screwdriver.

②

Hold the dispenser (a) above the back wall (e) against the wall and slide it down until it locks into place.

Plug the drip tray (g) onto the disinfection dispenser.

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③

Open the dispenser by pressing the device below the arm lever.

The disinfectant dispenser can be locked with the included key (i). To do this, the key (i) must be inserted into the opening provided for this purpose on the bottom of the dispenser casing. To open and close the disinfectant dispenser, the key (i) must be turned clockwise.

If the keyhole is in a vertical position, the dispenser is open. When the keyhole is in a horizontal position, the dispenser is closed.

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④

Insert the bottle (f) into the dispenser (a) from below.

To use the dispenser (a), hold your hand under the dispenser and operate the arm lever.

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