

MaiMed® MyClean FD

Dosiertabelle

Gebrauchslösung*	0,50%	1,00%	1,50%	2,00%	3,00%	4,00%
1 Liter	5 ml	10 ml	15 ml	20 ml	30 ml	40 ml
2 Liter	10 ml	20 ml	30 ml	40 ml	60 ml	80 ml
3 Liter	15 ml	30 ml	45 ml	60 ml	90 ml	120 ml
4 Liter	20 ml	40 ml	60 ml	80 ml	120 ml	160 ml
5 Liter	25 ml	50 ml	75 ml	100 ml	150 ml	200 ml
8 Liter	40 ml	80 ml	120 ml	160 ml	240 ml	320 ml
10 Liter	50 ml	100 ml	150 ml	200 ml	300 ml	400 ml

*Gebrauchslösung = Menge Desinfektionsmittelkonzentrat + Differenzmenge Wasser.

Beispiel 1L 1,00% Lösung



Einwirkzeiten

Wirksamkeiten	0,50%	1,00%	1,50%	2,00%	3,00%	4,00%
bakterizid & levurozid	5 Min.		5 Min.			
tuberkulozid		1 Std.		30 Min.		
mykobakterizid		1 Std.		30 Min.		
begrenzt viruzid					5 Min.	
begrenzt viruzid PLUS						1 Std.
Adenovirus Typ 5						1 Std.
Murines Norovirus					1 Std.	
Rotavirus Stamm Wa		30 Min.				
Viruzid (Polyomavirus SV40)				1 Std.		